




# PASSWELL COSY heat pad



## Birds & Animals

Boxes and Hospital Cages.  
Place pad underneath or  
inside box or cage.

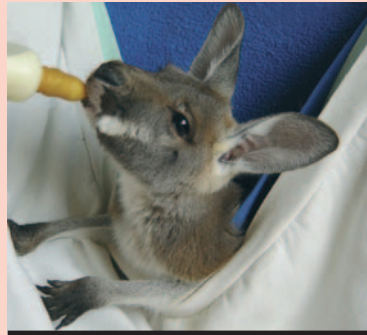

## Hanging Pouches

Place pad at back of pouch.  
Put insulation between pad  
and animal.



## Veterinarians

Use on operating tables and  
in recovery rooms.



## Dog & Cat Beds

Place underneath bedding and provide suitable  
protection from chewing.



## Reptiles

Place at one end of the enclosure so reptile can  
move on or off the heated area. Cover with layer  
of sand or flat rock.

## Using the Pad

The **Cosy Heat Pad** is designed to provide warmth for sick, young or  
debilitated animals and birds. The pad works by gradually heating up the  
area under where the animal sits.

The heater produces a temperature on the pad surface of about  
15 - 20°C above room temperature.

**When room temperature is less than 10°C** animals should be as close as  
practicable to the pad surface.

**When room temperature exceeds 10°C** animals should be insulated from  
the pad surface with layers of woollen fabric. The thickness of insulation  
required will depend on the room temperature and should be adjusted so  
that the animal is comfortable.

Monitor the temperature where animal rests with a thermometer. It  
should be from 20°C to 35°C, depending on type and age of animal. Seek  
professional advice if unsure.

**Heat stress can occur if the temperature is too high.**

This device is fitted with a cut-off switch which limits the surface  
temperature of the pad, preventing excessively high temperatures.

### SPECIFICATION

**Size:** 260mm x 360mm  
**Cover:** Soft non toxic PVC

**Volts:** 240 AC  
**Power:** 10 Watts

### Warranty on COSY Heat Pad

This device is warranted by the manufacturer, Passwell PTY LTD ACN 008 137 175  
("the Company") to be free from defects in materials and workmanship for a period of 12  
months following the date of purchase.

#### Conditions:

- During the warranty period the Company will repair or replace the heat pad.
- The Company will decide if there are any defects in material and/or workmanship.
- The warranty applies only for mainland Australia and Tasmania.
- The device must be operated in accordance with the Company's Instructions.
- It does not apply to damage caused by accident, alterations, misuse, fire or floods.
- It applies only to the original buyer and they complete and keep the data below.

Serial No.: \_\_\_\_\_ Purchase Date: \_\_\_\_\_

Purchased From: \_\_\_\_\_

CODE: PAD

